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INSTRUCTIONS FOR USE

Open the pouch along the indicated line.

1. Determine application sites. Remove any excessive hair, calluses and ensure the skin is clear of any oils, creams, fluids, dirt or other impediments to good electric contact.
2. If necessary, clean the skin with appropriate preparation material. Always ensure the application site is dry prior to application of electrodes.
3. Connect the lead wires to the electrodes.
4. Remove the plastic cover and position the electrodes on the skin. Smooth the backing surface with your finger as necessary.

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EC REP Medico Electrodes Ltd.
Blvd. Saint-Michel 65/6,
1040 Brussels, Belgium

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Area of Application / Contra indications

- For use in passive sensing of bio-electric signals from the intact skin.
- For single use only. Use on other patients can cause cross infection.
- Apply only under the supervision of a physician. Any allergic or other skin reactions should be immediately evaluated by a physician.
- The product is not for internal use. In case of accidental indigestion of the electrode consult a physician immediately. Not suitable for use on newborn or premature infants.
- Due to high adhesion strength of the adhesive material, this product is not suitable for use on patient with fragile skin.
- Use only in combination with approved patient cables and adapters. Never allow contact with an external electric source.
- In the event of use over an extended period, check for secure adhesion and absence of skin reaction on a daily basis. Electrodes may not be repositioned.

DO NOT USE IF THE GEL IS DRY OR DISCOLORED

Storage

- Store in original packaging protected from weather & between the indicated temperatures.
- Once a pouch is opened, use electrodes promptly. Protect remaining electrodes in original pouch from moisture loss by folding-over the top and securing.
- Dispose off as hospital contaminated waste, in accordance with regulations in force.
- If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

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