3M™ Red Dot™

Resting EKG Electrode

(English)

Intended Use and Product Description:

The information provided in this insert applies to the 3M Red Dot 2300, 7300 and 9600 Series of Resting EKG Electrodes with Tab Style Connector. These electrodes are disposable with a silver/silver chloride sensing element and are designed for a patient undergoing a short-term EKG diagnostic procedure while resting.

Warning:

This device has not been tested for use during Magnetic Resonance Imaging (MRI) procedures. The use of conductive patient-connected devices and patient lead wires/electrodes in MRI procedures may result in serious patient burns. If a physician orders this device to be used during MRI procedures, the user should not allow loops in patient lead wires or allow the lead wires to come in contact with the patient's skin.

Caution:

U.S.A. federal law restricts this device to sale by or on the order of the physician.

Precautions:

- Once the package has been opened, fold end to preserve freshness of the remaining electrodes.
- Do not use an electrode if the conductive adhesive is dry and no longer sticky.
- Do not apply electrodes over pantyhose.
- For proper skin management and to minimize skin irritation:
 - Avoid placing an electrode on an irritated skin site.
 - Do not abrade a skin site more than one time.
 - Avoid placing electrodes on skin still wet from an alcohol wipe (dry thoroughly).
- Do not use for long-term monitoring.

Explanation of Symbols:

-  • Attention, see instructions for use
-   2010-10 AZ • The lot in a box and the hourglass symbols are symbols that represent lot number and expiration date. The hourglass is followed by a year and month which represent the expiration date (year and month: 2010-10). The entire line after the hourglass represents the lot number (2010-10 AZ).
-  • Do Not Reuse

Instructions For Use:

1. Before Electrode Application:

- Select and prepare an electrode site according to your health care facility's protocol for resting EKG monitoring.
- The electrode site should be clean, dry and free of skin oil before application of the electrode to allow optimal electrode adhesion to skin. Application sites with heavy or curly hair that cannot be parted should be clipped.
- For best trace quality, skin abrasion should be part of the skin preparation. Use 3M Red Dot Trace Prep as part of the skin preparation.

2. Electrode Application:

- Remove electrode from the liner. Apply to skin at standard placement sites.
- With tabs pointing toward the patient's waist, firmly press electrodes to assure uniform contact.
- For difficult areas such as hairy sites or contours (chest), use more downward pressure to increase adhesion.
- Place all electrodes on patient before any clips are connected.
- Recommended placement order is limb lead sites, followed by chest lead sites.

3. Lead Wire/Clip Connection:

- Attach clip-on connector to tab portion of electrode without clipping into adhesive.
- Clip electrodes in the order of electrode application, limb leads first and then chest leads.
- Ensure that metal of clip is in contact with conductor side (underside) of electrode tab.
- Avoid lead wire tension on electrode tab. Position cable yoke to provide adequate lead wire slack. Rotate clip/adaptor on lead wire terminal pin to reduce tension.

Shelf Life:

For shelf life, refer to the expiration date that is printed on each package.

Thank you for choosing 3M Red Dot electrodes. If you are not completely satisfied or if you have questions or comments, in the U.S.A., please call the 3M Health Care Customer Helpline at 1-800-228-3957. Outside of the U.S.A., please contact your 3M Distributor or your 3M Sales Representative.

3M

Made in U.S.A. by

 3M Health Care
St. Paul, MN 51144-1000
(U.S.A.) 1-800-228-3957
www.3M.com

3M and Red Dot are trademarks of 3M
©3M 2003
Issue Date: 2003-09 34-8707-6216-7

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