Rehosoft[®]...NG

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ADVANTAGES:

- High stability.
- Ease of use and set up.
- Injected foam for optimal comfort

MANUAL:

Before any installation, it is necessary to make sure the toilet bowl is well maintained.

For the model with lid, do not remove the Rehosoft ... NG from its packaging by the cover

ASSEMBLY:

- Simply place the raised toilet seat on the toilet bowl. The Rehosoft ... NG fits perfectly and fits into a large majority of bowls.

The outstanding stability of the Rehosoft ... NG provides an optimal safety.

DESCRIPTION:

- Dimensions: W 38 x L 40.5 x H 14 cm
- User raising height : 11 cm
- Net weight without lid: 2.4 kg, net weight with lid: 3 kg
- Maximum use weight: 120 kg

COMPOSITION:

- Polyvinyl chloride (PVC) surface
- Polyurethane foam injected
- Polypropylene lid

CLEANING:

- Cleaning: Soapy water and all usual products
- Disinfection: Chlorine dilution, disinfection aerosol

PRECAUTIONS FOR USE:

- Do not immerse the rear of the Rehosoft ... NG
- The Rehosoft ... NG must be placed on a flat surface, do not leave, for example deodorants on the toilet bowl
- Avoid contact with colored products as much as possible

PharmaOuest Industries cannot be held responsible for any damage related to misuse of the product.

Rehosoft...NG

❶	References
❷	Designation
❸	Without lid
❹	With lid

ADVANTAGES:

- Comfortable and ergonomic:
 - a general slope of 3 ° of the seat respects the natural angulation of the pelvis - a curved upper profile follows the shape of the body
 - large support surfaces at the front and rear thanks to the special profile - easy-grip and ergonomic grip wheel
- Daily practice:
 - Large clearances at the front and rear to facilitate intimate hygiene - opening the cover without touching the seat
 - possibility of sitting with the lid folded down
- Hygienic product design:
 - no external roughness - easy to clean - liquids do not penetrate inside

INSTALLATION:

The toilet seat must be removed before attaching the product. The assembly is carried out by wedging the front shape of the riser against the front of the WC and then tightening the 2 fixing lugs using the side screws. The ergonomic and robust design of the screws allows for powerful tightening. The anti-slip treatment of the legs avoids the risk of slipping.

FITTING THE LID:

- The lid is assembled and removed by a simple snap in the inverted position
- In the folded down or vertical position, a safety device prevents the cover from latching

DESCRIPTION:

- Dimensions: 43 x 44.5 cm
- Maximum user weight: 180 kg

COMPOSITION: polypropylene

CLEANING:

- Cleaning of water, soap and non-abrasive household detergents
- Withstands machine washing and autoclave 120 ° C for 30 minutes
- Do not immerse, do not return the product during washing

PRECAUTIONS FOR USE:

- Do not climb on the product
- Check the tightness of the legs before each use
- The Rehotec must be placed on a flat surface, do not leave, for example deodorants on the toilet bowl
- Do not use this product if it has any defects or if it is broken
- Keep out of reach of children

PharmaOuest Industries cannot be held responsible for any damage related to misuse of the product.

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