*The natural way of Sweden*

# SISSEL®

## LINUM · VINO THERM

## CHERRY · CHERRY HEAT BELT

## BALU

- ▮ Gebrauchsanweisung
- ▮ Instructions for use
- ▮ Mode d'emploi
- ▮ Gebruiksaanwijzing
- ▮ Instrucciones de uso
- ▮ Istruzioni per l'uso
- ▮ Bruksanvisning
- ▮ 사용설명서

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## SISSEL® HEATING/COOLING PADS WITH NATURAL FILLING

### General information

Read the following instructions for use carefully and observe the safety instructions and warnings. SISSEL® heat/cold pads with natural filling are overlays for targeted, local heat

or cold therapy. As a medical device, it is primarily intended for use by healthcare professionals. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regard-

ding the diseases to be treated with the system or how to avoid them. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists.

SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use. The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

### Safety instructions

- ! Check the integrity of the product before each use. If damaged, the function of the product and safety are no longer guaranteed, escaping hot filling can cause skin burns.
- ! Heat the grain pillow in the microwave or in the oven. Do not heat the grain pillow in a water bath or grill.
- ! Keep the area of use free from sharp or pointed objects.
- ! Do not leave the microwave or oven unattended when heating the cushion. Overheating can destroy the cushion, there is a risk of fire! Follow the microwave manufacturer's or oven manufacturer's operating instructions carefully.
- ! Shake the grain pillow well after each

heating. This way the heat is evenly distributed in the filling and no heat pockets are created (danger of burning!).

- ! Never place the grain pillow directly on the skin! Always use a thin cotton cloth as an intermediate layer to avoid damage to the skin (e.g. frostbite, chilblains, burns).
- ! Check the temperature by gently holding the grain pillow against the inside of the wrist.
- ! Remove the grain pillow immediately if you feel uncomfortable with the cold or heat.
- ! Children or people with cognitive impairments should only use the grain pillows under supervision.
- ! Leaking, damaged or deflated grain pads must not be used. Danger of burns!  
Choking hazard!
- ! The contents of the grain pillow are not suitable for consumption.
- ! Only use accessories intended by the manufacturer.
- ! The product and its accessories are not toys and must be kept out of the reach of children. Do not allow children to play with it.

### Product information

Scope of delivery consisting of: 1x heat/cold pad with natural filling, 1x instructions for use

## Technical specifications

### SISSEL® Linum

Dimensions (LxW):	Classic:	45 x 30 cm
	Anatomic:	38 x 36 cm
	Cervico:	43 x 26 cm
Material:	Filling:	linseed
	Cover:	cotton

### SISSEL® Vinotherm

Dimensions (LxW):	35 x 19 cm	
Material:	Filling:	grape seeds
	Cover:	cotton

### SISSEL® Cherry

Dimensions (LxW):	23 x 26 cm	
	20 x 40 cm; heart-shaped	
Material:	Filling:	cherry stones
	Cover:	cotton

### SISSEL® Cherry Heat Belt

Dimensions (LxW):	50 x 17cm	
Material:	Filling:	Kirschkerne
	Cover:	cotton

### SISSEL® Balu

Dimensions (LxW):	16,5 x 24 cm	
Material:	Filling:	cherry stones
	Cover:	cotton

Note: The product is subject to natural weight fluctuations.

## Intended use

SISSEL® heating/cooling pads with natural filling are overlays for a drug-free, need-based, individual heat or cold therapy. Heat

and cold applications use the reactions of the skin, the subcutis, the musculature and deep tissue layers to the respective temperature stimulus. In heat therapy, these are primarily effects that promote circulation and stimulate the metabolism; in cold therapy, they are heat withdrawal and pain relief.

Thus, the grain pillow is an ideal way to use heat and cold in a targeted therapeutic way on various occasions.

## Indication heat

Tension, sore muscles, myogelosis/hard tension, back pain, tendopathies, muscle and tendon shortening, lumbago, menstrual cramps, myalgias, irritations of the gastrointestinal tract, hypertension (high blood pressure), general nervousness and restlessness, colds.

## Indication cold

Local tissue irritation (swelling, bruising), sports injuries, pain relief, traumatic and rheumatoid arthritis, haematomas (bruises), sprains (sprains), local scalds or burns, insect bites, headaches, migraines, nosebleeds, fever, tired and heavy legs.

## Contraindication heat

Acute inflammation, acute trauma (accidents, open wounds), acute joint inflammation, oedema, febrile infections, dermatitis, deep vein thrombosis, peripheral arterial oc-

clusive disease, hypotension, heart disease, impaired skin sensation (e.g. diabetics, persons insensitive to heat/cold, children under 3 years). If in doubt, seek medical advice.

### **Contraindication cold**

Cold intolerance and cold allergy, Raynaud's syndrome, circulatory disorders, acute trauma (accidents, open wounds), peripheral arterial occlusive disease, deep vein thrombosis, impaired skin sensation (e.g. diabetics, heat/cold insensitive persons, children under 3 years). If in doubt, seek medical advice.

### **Regulatory requirements**

The following regulatory requirements are fulfilled by this product of SISSEL® GmbH: Regulation (EU) 2017/745 on medical devices. All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

### **Usage**

Note:

- Check the packaging and the product for any damage on delivery.
- ⚠ Warning: Never use a damaged product.
- Check the product for integrity and contamination before each use.
- Never place the product directly on the skin, always use a thin cotton cloth as an intermediate layer.

- Use the product as intended.

### **Heat application**

Only heat the grain pillow in the microwave or oven.

### **Microwave:**

- Make absolutely sure that the plate with the grain pillow can rotate unhindered in the microwave and that there is no food residue in the microwave! Place the grain pillow in the microwave and distribute the filling evenly. The actual performance of different microwave brands varies despite the same power specifications. We therefore recommend that you try to find the optimum heating time. You can use the following information as a guide:
  - 600 Watt: 1 minute, repeat in 20 second increments
- Remove the grain pillow from the microwave and check the temperature. If further heating is required, shake the product and heat it again for 20 seconds. Then check the temperature again and heat further if necessary until the desired temperature is reached.

### **Oven:**

- Preheat the oven to 70°C (top and bottom heat or convection). Place the grain pillow flat on a clean rack, if necessary with baking paper and place in the oven for 4 to 8

minutes on the middle shelf.

- Carefully remove the grain pillow from the oven and check the temperature. If further heating is desired, shake the product and heat it again for 1 to 2 minutes. Then check the temperature again and heat further if necessary until the desired temperature is reached.

 **Warning:** Risk of injury! The heat may be unevenly distributed at first. It is important to shake the cushion well after each heating interval to distribute the temperature evenly in the filling.

 **Warning:** Risk of injury! Never place the grain pillow directly on the skin, always use a thin cotton cloth as an intermediate layer.

### **Cold application**

Place the grain pillow in a suitable plastic bag in the refrigerator or freezer/freezer for at least 2 hours. Place the grain pillow on the affected area with a thin cotton cloth as an intermediate layer. Only apply the grain pillow for as long as you find it comfortable - max. 20 minutes. Before using the grain pillow again, the treated area should remain uncovered for 20 minutes.

### **Care & Hygiene**

The present product is suitable for repeated use. However, personal application is recom-

mended, as classical treatment methods are not applicable to heating/cooling pads with natural filling. It is possible to hand wash the cherry pit cushions. However, the drying time in the air is several days. Improper drying can quickly lead to mould formation, so we advise against washing the product. Please always protect all grain pillows (linseed pillow, grape seed pillow) from moisture. Detergents, cleaning agents and disinfectants must not be used. For storage, protect the product from direct sunlight and store in a clean, dark, dry place when not in use.

### **Disposal & Environment**

- **Product:** The cover can be added to household waste (residual waste). The filling can be added to the organic waste (compost).
- **Packaging:** No special types of packaging are used. The respective valid disposal guidelines must be observed.

### **Warranty**

Operators of this product must comply with the regulatory requirements for medical devices applicable at the place of use. The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

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